GTS™

Great Toe Splint

For post-operative use after Hallux Valgus and other great toe joint conditions.

DARCO®

GTS-PKG-21_Rev0321

DARCO® (Europe) GmbH
 Gewerbegebiet 18
 82399 Raisting, Germany
 www.darco.de
 Made in China

Manufactured for:
 DARCO® International
 810 Memorial Blvd.
 Huntington, WV 25701 | USA
 Toll Free: 800-999-8866
 Phone: 304-522-4883
 www.darcointernational.com

GTS™ - Great Toe Splint

Size:	Avg. Shoe Size	Product Codes
S	W 5-7.5 / M 3.5-6	GT51R GT51L
L	W 8-11 / M 6.5-9.5	GT52R GT52L

Manufactured with lightweight neoprene and nylon, the GTS™ may be worn day and night while its form-fitting design easily accommodates suitable footwear.

› Maintains alignment

› 24 hour support

› Helps relieve painful bunions

› Can be worn post-operatively

GTS™ - Great Toe Splint

Instructions For Use

EN

Indications

> Hallux valgus and other great toe joint conditions

Contraindications

> No contraindications are known.

Cleaning Instructions/Maintenance

> When the GTS™ is slightly dirty, clean it with a damp cloth or a soft brush.
> Allow the GTS™ to air dry only.

Instructions for use

1. Place the toe cup over the big toe. Position connecting strap along the side of the foot.
2. Place your heel in the heel cup formed by the strap.
3. Apply the hook and loop strap snugly around the top of the foot. Secure strap to ensure hook and loop isn't rubbing the top of the foot.
4. Final adjustments may be made by easing the strap-A through slot-B and either pulling it tighter or looser as required.
5. Move the top strap forward to increase the tension or back to decrease it.

Caution

Please always follow these instructions for use. In case of pain or swelling, discontinue use and consult your medical provider immediately. This device is designed to assist recovery and should be used exclusively under the instruction of and in consultation with a healthcare professional. DARCO® International Inc. does not assume any responsibility for misuse of the product. It is advisable to wear dressings or socks with the GTS™. Do not drive a vehicle while you are wearing this device. Use extreme caution when walking on all surfaces when using this device. Walk more slowly than normal. This device is for single-patient use only.

To the patient/user: This device has been designed and manufactured by DARCO® International, Inc. to be state of the art for treatment of a specific medical condition. Should any serious incident occur in relation to this device it should be reported to the manufacturer at the contact information listed on this document and the proper authority of the country or state where you are located. When treatment is complete, dispose of this device according to local laws and ordinances.

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