PegAssist™ Insole Instructions



Place a transparent barrier or dressing over the wound area.



With the barrier in place, circle the wound or ulcer on the patient foot with the supplied marker.



Remove the black insole from the shoe.



Insert the PegAssist™ Insole into the shoe with the **peg side facing down**.



Have the patient step onto the insole which will transfer the marking onto the top surface of the insole. (Remove barrier)



Remove the insole from the shoe. Remove pegs beneath the marking to create the off-loading area.

Important: Remove the pegs slowly to avoid tearing the top layer of PORON®.



Remove the adhesive backing from the stabilizer board that is included in the kit



Align and attach the adhesive side of the stabilizer board to the **peg side** of the insole.



Insert the PegAssist™ Insole back into the shoe with the stabilizer board facing down, blue side up. The shoe is now ready to be worn.

Caution:

Use of this product is limited to single-patient use, under order of a physician or licensed medical professional. If you experience additional pain, swelling or skin irritation, discontinue use and contact your physician immediately.