Indications

- Casted fracture
- Bulky bandages

Contraindications

- Open fractures

Cleaning Instructions/Maintenance

- When the shoe is slightly dirty, clean it using a damp cloth or a soft brush.
- Allow the shoe to air dry only.

Instructions for use

1. Unhook the bungee-closure system.
2. Press and hold the fastener of the locking mechanism and slide toward the toes in order to widen the ankle strap.
3. Place foot in cast shoe.
4. While holding the end of the bungee-closure system, press and hold the fastener of the locking mechanism and slide toward the ankle.
5. Hook the bungee-closure system.

Caution

Please always follow these instructions for use. In case of pain or swelling, discontinue use and consult your medical provider immediately. This device is designed to assist recovery and should be used exclusively under the instruction of and in consultation with a healthcare professional. DARCO International Inc. does not assume any responsibility for misuse of the product. Do not drive a vehicle while you are wearing this device. Use extreme caution when walking on all surfaces when using this device. Walk more slowly than normal. This device is for single-patient use only.

To the patient/user: This device has been designed and manufactured by DARCO International, Inc. to be state of the art for treatment of a specific medical condition. Should any serious incident occur in relation to this device it should be reported to the manufacturer at the contact information listed on this document and the proper authority of the country or state where you are located. When treatment is complete, dispose of this device according to local laws and ordinances.

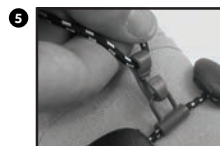
Manufactured for:
DARCO International, Inc.
810 Memorial Blvd.
Huntington, WV 25701, USA
Phone 304.522.4883

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