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**Manufactured for:**  
DARCO® International, Inc.  
810 Memorial Blvd.  
Huntington, WV 25701, USA  
Phone 304.522.4883

[www.darcointernational.com](http://www.darcointernational.com)

**UK Responsible Person:**  
V-M Orthotics Ltd.  
Unit 25 Halesworth Business  
Centre Norwich Road  
Halesworth Suffolk IP19 8QJ  
United Kingdom

Made in China

DARCO® (Europe) GmbH  
Gewerbegebiet 18  
82399 Raisting, Germany  
Telefon +49 8807 9228 0  
[www.darco.de](http://www.darco.de)

BATG-IFU-21\_Rev0225

# Body Armor® Toe Guard

## Instructions For Use

**Warning: This product has not been evaluated for use by OSHA for jobsite impact protection.**

#### **Indications**

- > Postoperative healing
- > Trauma care
- > Presence of K-wire
- > Diabetic foot/neuropathy

#### **Contraindications**

- > Not intended as a worksite protective device

#### **Cleaning Instructions/Maintenance**

- > When the toe guard is slightly dirty, clean it with a damp cloth or a soft brush.
- > Allow the toe guard to air dry only.

#### **Instructions for use: Body Armor® Toe Guard - Square Toe**

1. Open the shoe by means of the closures.
2. Remove insole.
3. Remove the white adhesive protective sheet on the underside of toe guard.
4. Place toe guard adhesive side down on the polyboard base of shoe. Ensure the toe guard is flush with the front lip of the shoe.
5. Press toe guard into shoe and hold for 10 seconds.
6. Place insole back into shoe.
7. Secure the foot in the shoe with the closures.
8. The toe guard should not move or slide while walking.

#### **Instructions for use: Body Armor® Toe Guard - Round Toe**

1. Open the shoe by means of the closures.
2. Trim Toe Guard - Round Toe to size, if needed.
3. Remove the white adhesive protective sheet on the underside of toe guard.
4. Place toe guard adhesive side down on the insole. Ensure the toe guard is flush with the front of the insole.
5. Press Toe Guard into shoe and hold for 10 seconds.
6. Secure the foot in the shoe with the closures.
7. The Toe Guard should not move or slide while walking.

#### **Caution**

Please always follow these instructions for use. In case of pain or swelling, discontinue use and

consult your medical provider immediately. This device is designed to assist recovery and should be used exclusively under the instruction of and in consultation with a healthcare professional. DARCO® International Inc. does not assume any responsibility for misuse of the product. It is advisable to wear dressings or socks with the Body Armor® Toe Guard. Do not drive a vehicle while you are wearing this device. Use extreme caution when walking on all surfaces when using this device. Walk more slowly than normal. This device is for single-patient use only.

*To the patient/user: This device has been designed and manufactured by DARCO® International, Inc. to be state of the art for treatment of a specific medical condition. Should any serious incident occur in relation to this device it should be reported to the manufacturer at the contact information listed on this document and the proper authority of the country or state where you are located. When treatment is complete, dispose of this device according to local laws and ordinances.*

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