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**UK
CA**

Manufactured for:
DARCO® International, Inc.
810 Memorial Blvd.
Huntington, WV 25701, USA
Phone 304.522.4883

www.darcointernational.com

Made in China

CE

EC REP

DARCO® (Europe) GmbH
Gewerbegebiet 18
82399 Raisting, Germany
Telephone +49 8807 9228 0
www.darco.de

BWAL-IFU-21_Rev0225

DARCO

Body Armor® Walker

Instructions For Use

| Body Armor® Walker

| Body Armor® Walker Liner

EN

| Body Armor® Walker II

Indications

> Postoperative use, trauma care, stress fractures, soft tissue injury

Contraindications

> Open fractures, poor circulation

Cleaning Instructions/Maintenance

- > When the shell or inner liner is slightly dirty, clean it with a damp cloth or a soft brush.
- > Allow the shell or inner liner to air dry only.

Instructions for use - Body Armor® Walker

- Unlock all buckles and remove the shin guard, foam pads, and liner.
- Wrap the liner around the foot and secure with the closures - *Additional Liners are available.*
- Spread open the shell and place the foot heel first into the walker.
- Place the foam pads in areas of excessive rubbing or to limit space (Optional).
- Place the shin guard into position and secure it under the edges of the walker shell.
- Insert straps into the buckles and adjust them to the desired compression. Close the blue-tipped buckles to the locked position.

Instructions for use - Body Armor® Walker II

- Unlock all buckles, remove the shin guard and foam pads.
- If off-loading for ulcers, fractures, or other painful areas, see below instructions.
- Spread open the shell and place the foot heel first into the walker.
- Place the foam pads in areas of excessive rubbing or to limit space (Optional).
- Place the shin guard into position and secure it under the edges of the walker shell.
- Insert straps into the buckles and adjust them to the desired compression. Close the blue-tipped buckles to the locked position.

Off-load site modification - Body Armor® Walker II

- Remove top insole (tan) from the walker.
- Place a sterile film over the area to be off-loaded.
- Mark the area to be off-loaded on the film with a transferable marker.
- Place the foot in the walker and stand to allow the mark to transfer to the bottom insole (white). Remove bottom insole when complete.
- Using a utility knife or rotary tool, cut out the relief area. Skive the edges (approx. 30°). The relief area should be 0.5 cm larger than the outer edge of the area to be offloaded and extended distally 1.25 cm.
- Place the modified bottom insole back in the walker and cover with the top insole.

Off-load site management should be under the instruction of and in consultation with a healthcare professional.

Caution

Please always follow these instructions for use. In case of pain or swelling, discontinue use and consult your medical provider immediately. This device is designed to assist recovery and should be used exclusively under the instruction of and in consultation with a healthcare professional. DARCO® International Inc. does not assume any responsibility for misuse of the product. It is advisable to wear dressings or socks with the walker. Do not drive a vehicle while you are wearing this device. Use extreme caution when walking on all surfaces when using this device. Walk more slowly than normal. This device is for single-patient use only.

To the patient/user: This device has been designed and manufactured by DARCO® International, Inc. to be state of the art for treatment of a specific medical condition. Should any serious incident occur in relation to this device it should be reported to the manufacturer at the contact information listed on this document and the proper authority of the country or state where you are located. When treatment is complete, dispose of this device according to local laws and ordinances.

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