Indications

- For treatment of diabetic foot syndrome
- To facilitate the healing of open wounds and minor ulceration
- For pressure off-loading
- Up to Wagner grade 3 ulcerations
- Bony prominences

Contraindications

- Open fractures
- Ulcerations greater than Wagner grade 3

Cleaning Instructions/Maintenance

- When the shoe or insole (top layer only) is slightly dirty, clean it with a damp cloth or a soft brush.
- Allow the shoe or insole to air dry only.

Instructions for use

1. Apply a transparent dressing (*not included*) over the wound.
2. Circle the wound with the wound marker (*included*).
3. Place the insole (*blue side up*) inside the shoe. This will be a snug fit. Have the patient step on the insole to transfer the marking.
4. Take out the insole and remove the pegs beneath the marking.
5. Peel the adhesive backing from the stabilizer board (*included*).
6. Attach the stabilizer board to the peg-side of the insole. Reinsert the insole into the shoe (*blue side up*).
7. Place the foot in the shoe with the heel firmly seated, and adjust the fit with the hook and loop straps.

Caution

Please always follow these instructions for use. In case of pain or swelling, discontinue use and consult your medical provider immediately. This device is designed to assist recovery and should be used exclusively under the instruction of and in consultation with a healthcare professional. DARCO International Inc. does not assume any responsibility for misuse of the product. It is advisable to wear dressings or socks with the shoe. Do not drive a vehicle while you are wearing this device. Use extreme caution when walking on all surfaces when using this device. Walk more slowly than normal. This device is for single-patient use only.

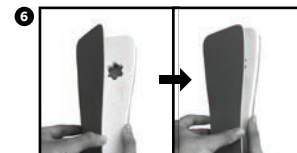
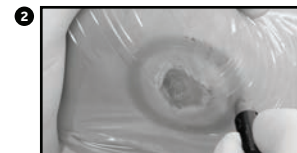
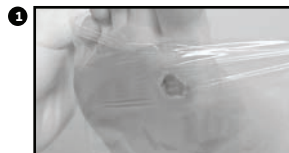
To the patient/user: This device has been designed and manufactured by DARCO International, Inc. to be state of the art for treatment of a specific medical condition. Should any serious incident occur in relation to this device it should be reported to the manufacturer at the contact information listed on this document and the proper authority of the country or state where you are located. When treatment is complete, dispose of this device according to local laws and ordinances.

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