AllRound Shoe[®]

- EN** Closed long-term Healing Shoe
- DE** Geschlossener Verbandschuh
- ES** Zapato postquirúrgico cerrado larga duración
- FR** Chaussure fermée pour pansement longue durée
- IT** Calzatura ortopedica chiusa per uso prolungato
- NL** Gesloten langetermijnverbandshoer
- DA** Lukket bandagesko til langtidsanvendelse
- SV** Slutet läkningsko för långvarig användning
- PL** Zamknięty długoterminowy but leczniczy
- SL** Zaprt terapevtski čevlji
- HR** Zatvorena ortopedska cipela za dugoročnu uporabu
- PT** Sapato fechado para terapia de longa duração

DARCO

AllRound Shoe®

AllRound Shoe® | Closed long-term Healing Shoe

Important notes

If there is a serious deterioration in your state of health as a result of using the medical device, please report this to your specialist dealer or the manufacturer as soon as possible. If you report this to the specialist dealer, the dealer will forward this immediately. The reports are agreed with and processed by the competent authorities.

Please always follow these instructions for use. In case of adverse effects (e. g. skin swelling, pressure points, disorders of blood circulation etc.) please tell your doctor. Function and optimum protection are no longer guaranteed if the product is handled incorrectly. This aid assists recovery and should be used exclusively under the instruction of and in consultation with your doctor or a healthcare professional. DARCO (Europe) GmbH does not assume any responsibility. Do not wear the product with injured, irritated, damaged or intact skin. You should use bandages or socks, for example.

You must not drive a vehicle while you are wearing this product. Use extreme caution when climbing stairs (e.g. also stepping up onto curbs) and walking on uneven surfaces. Be particularly cautious when walking in wet, slippery and snowy conditions. Walk more slowly than usual.

Note: This device is allowed for single-patient use only. It does not need to be cleaned or reprocessed between uses.

Purpose

The AllRound Shoe® Healing Shoe provides stability, protects the foot against bumps and reduces pressure on it following foot surgery and other types of wounds on the foot.

Maintenance

The product does not require maintenance.

Indications / Contraindications

Indications

- For postoperative care
- For wound care
- Arthritis

Contraindications

No contraindications are known.

Design features

The upper is made of breathable nylon mesh that can be stretched and adjusted lengthways and sideways. The outer sole is made from EVA / rubber with a shock absorbing effect. The sole profile facilitates the rolling off movement. The base of the shoe is reinforced throughout. Other features include the wide opening, individually adjustable fastener, a removable sole as well as weatherproof durable material on the shank and sole.

Use

1. Open the shoe using the hook and loop straps (A, B).
2. Please ensure that the foot is securely positioned in the shoe.
3. Close the shoe using the hook and loop straps so that it is comfortable for the patient.

Working life / Lifespan of the product

The lifespan of the medical device is determined by natural wear and tear due to proper handling / cleaning. The medical device is designed to be used for the usual duration of use prescribed by a doctor without showing a reduction in safety or performance. After the period of use prescribed by your doctor, check the function of the product together with your doctor or specialist dealer.

Combination with other products

PegContour® Insole Uni

Technical data

Sizes

6 sizes: XS (37 – 38), S (39 – 40), M (41 – 42), L (43 – 44), XL (45 – 46), XXL (47 – 48), XXXL (49 – 50)

Colour

Black

Material

Outsole: Rubber, EVA

Insole: EVA

Upper material: Foam polyester mesh

Inner lining: Brushed polyester

Hook and loop straps: Nylon

Care instructions

Hand wash

- If the shoe is slightly soiled, you can wipe it with a damp cloth or clean it with a soft brush.
- Do not dry the shoe in a dryer or on a radiator. Leave it to dry at room temperature.

Disposal

Dispose of after use. Please dispose of the packaging in an environmentally friendly manner. Dispose of this product after use in accordance with applicable local laws and regulations.

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ALBO-Healthcare GmbH
V-M Orthotics Ltd.