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**Manufactured for:**  
DARCO International, Inc.  
810 Memorial Blvd.  
Huntington, WV 25701, USA  
Phone 304.522.4883

[www.darcointernational.com](http://www.darcointernational.com)

**UK Responsible Person:**  
V-M Orthotics Ltd.  
Unit 25 Halesworth Business  
Centre Norwich Road  
Halesworth Suffolk IP19 8QJ  
United Kingdom

DARCO (Europe) GmbH  
Gewerbegebiet 18  
82399 Raisting, Germany  
Telefon +49 8807 9228 0  
[www.darco.de](http://www.darco.de)

NSW-IFU-21\_Rev0625

# Night Splint

## Instructions For Use

### | Pneumatic Night Splint

### | Night Splint Wedge

#### **Indications**

- Plantar fasciitis
- Heel spurs
- Achilles tendon injuries
- Ankle contracture

#### **Contraindications**

- Plantar flexion contracture greater than 10°
- Uncontrollable severe plantar flexion spasticity
- Extended weight bearing and ambulation

#### **Cleaning Instructions/Maintenance**

- When the cover is slightly dirty, clean it with a damp cloth or a soft brush.
- Allow the cover to air dry only.

Remove the wedges before cleaning (Night Splint Wedge)

#### **Instructions for use - Pneumatic Night Splint**

1. Open all straps and place foot in the Pneumatic Night Splint with the ankle positioned at 90°.
2. Close the middle strap in order to hold the heel in place.
3. Extend the knee to test the fitting of the middle strap. If the heel is slipping inside the splint, the middle strap is too loose. If the calf is squeezed in, the middle strap is too tight.
4. After checking the correct fit of the middle strap, close the remaining straps.
5. To inflate, compress the inflate bulb multiple times until the desired angle is achieved. Caution: **DO NOT OVERINFLATE.**
6. To deflate, lift the cap, and press the release valve. Failure to deflate before weight bearing will result in irreparable damage to the air bladder.

#### **Instructions for use - Night Splint Wedge**

1. Open the toe flap and remove the two wedges from the wedge pocket.
2. Open all straps and place foot in the Night Splint Wedge with the ankle positioned at 90°.
3. Close the middle strap in order to hold the heel in place.
4. Extend the knee to test the fitting of the middle strap. If the heel is slipping inside the splint, the middle strap is too loose. If the calf is squeezed in, the middle strap is too tight.
5. After checking the correct fit of the middle strap, close the remaining straps.
6. Use 1 or 2 wedges [Under direction of a physician or healthcare professional].
7. Open the toe flap and insert wedge(s), thin end first.

#### **Caution**

Please always follow these instructions for use. In case of pain or swelling, discontinue use and consult your medical provider immediately. This device is designed to assist recovery and should

be used exclusively under the instruction of and in consultation with a healthcare professional. DARCO International Inc. does not assume any responsibility for misuse of the product. It is advisable to wear dressings or socks with the night splint. Do not drive a vehicle while you are wearing this device. Use extreme caution when walking on all surfaces when using this device. Walk more slowly than normal. This device is for single-patient use only.

*To the patient/user: This device has been designed and manufactured by DARCO International, Inc. to be state of the art for treatment of a specific medical condition. Should any serious incident occur in relation to this device it should be reported to the manufacturer at the contact information listed on this document and the proper authority of the country or state where you are located. When treatment is complete, dispose of this device according to local laws and ordinances.*

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