

- Ankle and knee pain
- Arch pain
- Plantar fasciitis/heel pain
- Over-pronation

Full Length PORON® Cushioning Insoles provide relief from:

Orthotic Insoles

ProMotion® Plus

DARCO ProMotion® Plus Orthotic Insoles:

- The soft high filament count polyester top fabric contains a permanent anti-microbial for odor control and low friction, allowing freedom of movement without socks bunching up.
- Full length PORON® Cushioning foam layer provides superior shock absorption
- Heel to toe cushioning foam that won't flatten or bottom out. Superior feel and long term wear benefits.
- EVA covered polypropylene heel and arch support. Provides a firm support that cradles the heel and arch area, The EVA surface prevents the insole from shifting while walking and adds durability.

Indications: Over-pronation, plantar fasciitis/heel pain, foot and arch pain, ankle and knee pain.

Preferred Shoe Type: Dress, casual, and athletic shoes.


Directions: Remove existing insoles from shoes and replace with Darco ProMotion Plus Insoles.

Care Instructions: Wipe with a damp cloth. Do not submerge in water. Allow insoles to air dry.

Warning: Always consult a medical professional if persistent pain, discomfort or swelling continues

Not intended for severe foot deformities or circulation problems, infections or foot ulcers.

Suggested Code: L3040

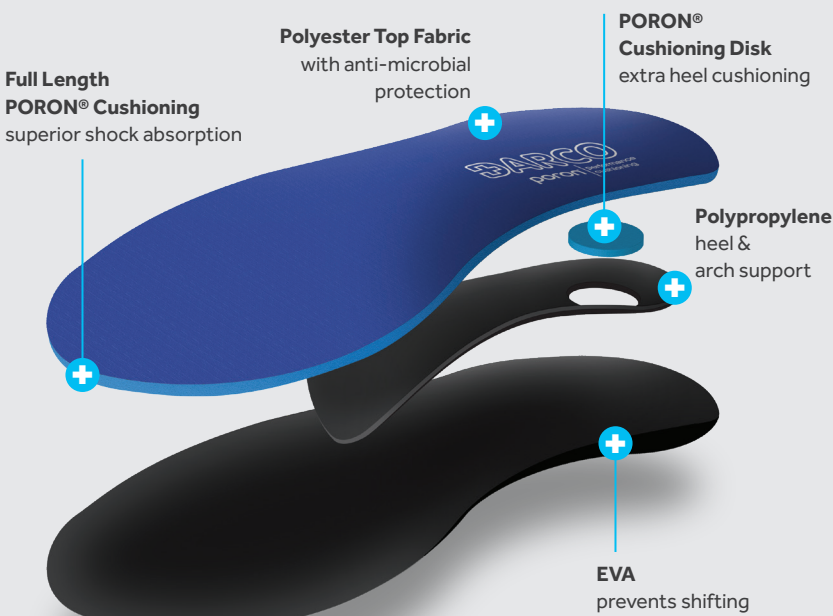

PORON is a licensed trademark of Rogers Corporation.

poron | performance cushioning

Manufactured for:

DARCO International, Inc.
810 Memorial Blvd.
Huntington, WV 25701, USA
Toll-free: 800.999.8866
Phone: 304.522.4883
www.darcointernational.com

Made in China

PMP-PKG-21_Rev0425_3-8