TAS™

Toe Alignment Splint

For post-operative use after Hallux Valgus, Hammer Toes, and Tailor's Bunion procedures.

DARCO

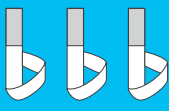
TAS-PKG-21_Rev0425


Made in China
DARCO (Europe) GmbH
Gewerbegebiet 18
82399 Raisting, Germany
Tel. +49 (0) 88 07. 92 28-0
www.darco.de

Manufactured for:
DARCO International, Inc.
810 Memorial Blvd.
Huntington, WV 25701, USA
Toll-free: 800.999.8866
Phone: 304.522.4883
www.darcointernational.com

Small toe loops x3

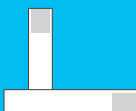Large toe loop

Heel band

Metatarsal band

Contents:

Scan QR code or visit:
darcointernational.com/tas

Video demonstration:
How to properly apply the TAS™

TAS™ Sizing
Size: Universal
Product Code: TAS

- Low profile metatarsal band fits easily into most footwear and reduces slippage.
- Toe loops gently guide toe(s) into proper alignment for post-procedural healing.
- Soft toe loops are less abrasive.
- Latex Free.

The TAS™ is an effective and inexpensive way of maintaining correct alignment of the toes during the healing process. A thin elastic metatarsal band reduces slippage and the soft toe loops hold the toes in the correct position to maintain alignment following surgery to correct hallux valgus, hammer toes or Tailor's bunions.

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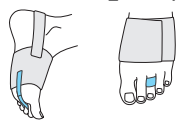
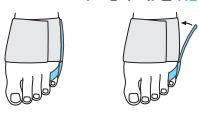
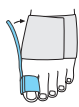
Contraindications

No contraindications are known.

Cleaning Instructions/Maintenance

- When the TAS™ is slightly dirty, clean it with a damp cloth or a soft brush.
- Allow the TAS™ to air dry only.

Instructions for use

The TAS™ is used for post-operative use to maintain the surgical result.

1. Fasten the metatarsal band securely around the foot just behind the metatarsal heads.
2. Secure the heel band by placing around the back of the heel and attaching to the medial and lateral sides of the metatarsal band.
3. For the proper attachment of the large toe loop and small toe loops, please refer to the illustrations above.

Caution

Please always follow these instructions for use. In case of pain or swelling, discontinue use and consult your medical provider immediately. This device is designed to assist recovery and should be used exclusively under the instruction of and in consultation with a healthcare professional. DARCO International Inc. does not assume any responsibility for misuse of the product. It is advisable to wear dressings or socks with the shoe. Do not drive a vehicle while you are wearing this device. Use extreme caution when walking on all surfaces when using this device. Walk more slowly than normal. This device is for single-patient use only.

To the patient/user: This device has been designed and manufactured by DARCO International, Inc. to be state of the art for treatment of specific medical condition. Should any serious incident occur in relation to this device it should be reported to the manufacturer at the contact information listed on this document and the proper authority of the country or state where you are located. When treatment is complete, dispose of this device according to local laws and ordinances.

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