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EN

Indications

> Postoperative use, trauma care, stress fractures, soft tissue injury

Contraindications

> Open fractures, poor circulation

Cleaning Instructions/Maintenance

> When the shell or inner liner is slightly dirty, clean it with a damp cloth or a soft brush.
> Allow the shell or inner liner to air dry only.

Instructions for use - FX Pro™ and HD Walker

1. Open the hook and loop straps, remove the inner liner of the walker and take out the foam pads.
2. Wrap the liner around the foot and secure with the closures.
3. Spread open the stirrups and place the foot heel first into the walker.
4. With leg in walker and ankle positioned at 90°, affix the stirrups to the center of the liner on each side.
5. Place the foam pads in areas of excessive rubbing or to limit space (Optional).
6. Begin securing straps, starting at the toes and working towards the knee.

Instructions for use - Air Traveler™

1. Open the hook and loop straps, remove the inner liner of the walker and take out the foam pads.
2. Wrap the liner around the foot and secure with the closures.
3. Spread open the stirrups and place the foot heel first into the walker.
4. With leg in walker and ankle positioned at 90°, remove the white protective strip and affix the stirrups to the center of the liner on each side.
5. Place the foam pads in areas of excessive rubbing or to limit space (Optional).
6. Begin securing straps, starting at the toes and working towards the knee.
7. To inflate, compress the inflate bulb multiple times until the desired angle is achieved. Caution: DO NOT OVERINFLATE.
8. To deflate, lift the cap, and press the release valve. Failure to deflate before weight bearing will result in irreparable damage to the air bladder.

Caution

Please always follow these instructions for use. In case of pain or swelling, discontinue use and consult your medical provider immediately. This device is designed to assist recovery and should be used exclusively under the instruction of and in consultation with a healthcare professional. DARCO® International Inc. does not assume any responsibility for misuse of the product. It is advisable to wear dressings or socks with the walker. Do not drive a vehicle while you are wearing this device. Use extreme caution when walking on all surfaces when using this device. Walk more slowly than normal. This device is for single-patient use only.

Manufactured for:
DARCO® International, Inc.
810 Memorial Blvd.
Huntington, WV 25701, USA
Phone 304.522.4883

www.darcointernational.com

**UK
CA**

UK Responsible Person:
V-M Orthotics Ltd.
Unit 25 Halesworth Business
Centre Norwich Road
Halesworth Suffolk IP19 8QJ
United Kingdom

Made in China

CE

EC REP

DARCO® (Europe) GmbH
Gewerbegebiet 18
82399 Raisting, Germany
Telefon +49 8807 9228 0
www.darco.de

WAL-IFU-23_Rev0225

to be state of the art for treatment of a specific medical condition. Should any serious incident occur in relation to this device it should be reported to the manufacturer at the contact information listed on this document and the proper authority of the country or state where you are located. When treatment is complete, dispose of this device according to local laws and ordinances.

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