Indications

- For treatment of diabetic foot syndrome
- For healing of open wounds and minor ulceration
- For pressure off-loading
- Postoperative after toe and forefoot surgery
- Up to Wagner grade 3 ulcerations
- Bony prominences

Contraindications

- Open fractures
- Ulcerations greater than Wagner grade 3

Cleaning Instructions/Maintenance

- When slightly dirty clean the shoes with a damp cloth or a soft brush. Do not use strong solvents for cleaning (alcohol etc.). Allow to air dry only.
- The insoles can be cleaned with a damp cloth or disinfectant wipe. Allow to air dry completely.

Instructions for use**Modifications are to be performed by a medical professional ONLY.**

The wound care shoes are used in pairs. Determining the correct shoe size is important for the perfect functionality of the shoe. The special fitting mark **B** located on the side of the sole assists in the proper fitting of the shoe. The ball of the foot or big-toe joint should be located at or just behind this marker. The WCS™ system supports patients limited in their mobility. For a proper stabilization of the foot the hook and loop straps should be closed securely. The removable insoles are included within the shoe (fig. 1) and additional insoles are available. The four-layer-insoles can be customized individually (fig. 2).

Marking the ulceration and customizing the insoles (weight-bearing ulcers):

1. Place a sterile film over the wound or pressure ulcer (fig. 3) and draw a circle on the film around the affected area with a marker pen (fig. 4).
2. The patient should now stand up in the shoe(s) onto the insoles, transferring the marking onto the upper layer of the insole.
3. Using the diagrams provided, determine the layers to be cut (see Insole placement and cutting). Using the upper layer as a guide, puncture through all layers to be cut with a straight pin (fig. 5).
4. Draw the areas to be removed onto the lower side of the layers as described in fig. 6.
5. Using a utility knife or rotary tool, cut out the relief area. Skive the edges (approx. 30°). The relief area should be 0.5cm larger than the outer edge of the area to be offloaded and extended distally 1.25 cm (fig. 7).
6. When cutting is complete, rearrange layers and place back in shoe.

Leather modification (nonweight-bearing ulcers):

1. To remove leather without damaging the liner, pass a blunt object (i.e. paperclip) from one perforated hole to the next. Be careful not to puncture the liner.
2. Using the blunt object as a guide, cut the leather.
3. Insert a flat instrument between leather and liner to continue cutting. Only cut within the perforated holes provided.

Insole placement and cutting (Refer to diagram A-E):

- A No plantar open ulcers
- B Level of ulceration 0 and 1, Wagner Scale, open wound
- C After wound healing (of B)
- D Level of ulceration 2 and up, Wagner Scale, open wound
- E After wound healing (of D)
- F Cross section depicting offloading lesion under third metatarsal head

Caution

Modifications are to be performed by a medical professional ONLY. Please always follow these instructions for use. In case of pain or swelling, discontinue use and consult your medical provider immediately. This device is designed to assist recovery and should be used exclusively under the instruction of and in consultation with a healthcare professional. DARCO International Inc. does not assume any responsibility for misuse of the product. It is advisable to wear dressings or socks with the WCS™. Do not drive a vehicle while you are wearing this device. Use extreme caution when walking on all surfaces when using this device. Walk more slowly than normal. This device is for single-patient use only.

To the patient/user: This device has been designed and manufactured by DARCO International, Inc. to be state of the art for treatment of a specific medical condition. Should any serious incident occur in relation to this device it should be reported to the manufacturer at the contact information listed on this document and the proper authority of the country or state where you are located. When treatment is complete, dispose of this device according to local laws and ordinances.

Manufactured for:
DARCO International, Inc.
810 Memorial Blvd.
Huntington, WV 25701, USA
Phone 304.522.4883

www.darcointernational.com

UK Responsible Person:
V-M Orthotics Ltd.
Unit 25 Halesworth Business
Centre Norwich Road
Halesworth Suffolk IP19 8QJ
United Kingdom

DARCO (Europe) GmbH
Gewerbegebiet 18
82399 Raisting, Germany
Telefon +49 8807 9228 0
www.darco.de

WCS-IFU-21_Rev0625


WCS™ Open-Toe

Modifications are to be performed by a medical professional ONLY.

**Plantar layer system:**

Material	Density	Thickness	Durometer*
Plastazote	soft	6.3 mm	26
EVA	medium	6.3 mm	42
EVA	firm	6.3 mm	50
Poron	high	3.1 mm	29

*The insole durometers may vary within 2° to 3° of those listed.

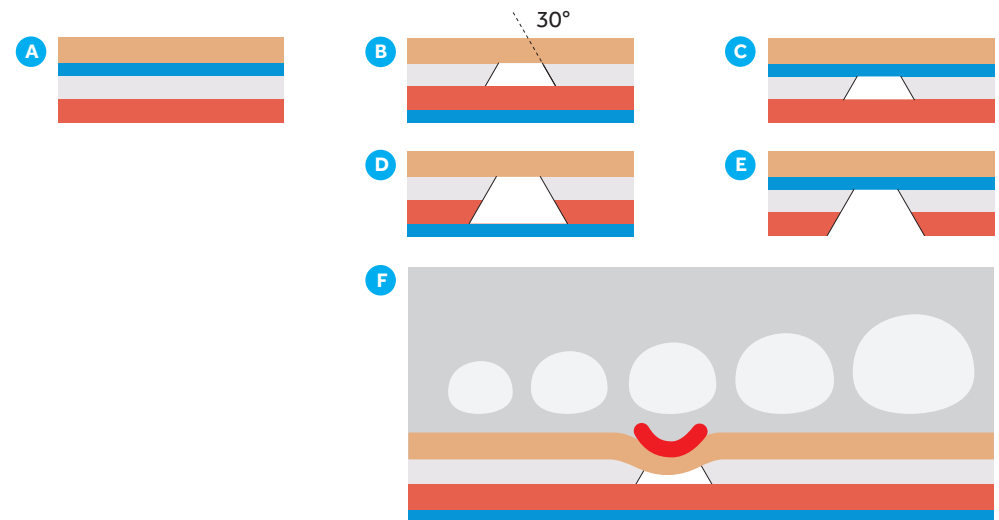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