DCS™

Plantar Fasciitis Sleeve

DARCO

DCS-PKG-24_Rev0825

Made in Taiwan
DARCO (Europe) GmbH
Gewerbegebiet 18
82399 Raisting, Germany
Tel. +49 (0) 88 07. 92 28-0
www.darco.de

Manufactured for:
DARCO International, Inc.
810 Memorial Blvd.
Huntington, WV 25701, USA
Toll-free: 800.999.8866
Phone: 304.522.4883
www.darcointernational.com

Instructions for use located on inside.

Available in 3 colors:

XL	11 - 15 in	-	13.5+	DCS-PF4	DCS-PF4-B	DCS-PF4-T
L	9 - 13 in	11+	10 - 13	DCS-PF3	DCS-PF3-B	DCS-PF3-T
M	7 - 10 in	7 - 10.5	6 - 9.5	DCS-PF2	DCS-PF2-B	DCS-PF2-T
S	5 - 8 in	4 - 6.5	3 - 5.5	DCS-PF1	DCS-PF1-B	DCS-PF1-T

Size: Arch Circ. Women: Men: Blue: Black: Tan:

Product Codes:

DCS™ Sizing (limited U.S.A. availability)

- › Wave compression technology provides a sustained stretch to the plantar fascia.
 - › Comfort Zone - Special knitting pattern at the Anterior Ankle Fold to help prevent pinching and binding.
 - › 24 hour support - wear while sleeping or under socks when active.
 - › Constructed of a medical grade fabric that is latex free.
 - › Anatomically designed for better comfort.
 - › Improves circulation and reduces edema.
- The DARCO DCS™ Plantar Fasciitis Sleeve is woven to be anatomically correct. Wave compression technology on the plantar surface of the foot provides a sustained stretch on the plantar fascia. The DCS™ gives relief from the discomfort associated with plantar fasciitis by promoting circulation, reducing edema and foot fatigue, and relieving morning foot pain.

DCS™ - Plantar Fasciitis Sleeve

Indications

- Plantar fasciitis, heel spurs, arch pain, arch weakness, edema

Contraindications

- Poor circulation, diabetic foot/neuropathy

Cleaning Instructions/Maintenance

- When the DCS™ is slightly dirty, clean it with a damp cloth or a soft brush.
- Allow the DCS™ to air dry only.

Instructions for use

- The top opening is indicated with the DARCO logo.
- Pull the sleeve over the foot until the heel is properly positioned in the heel pocket.

Caution

Please always follow these instructions for use. In case of pain or swelling, discontinue use and consult your medical provider immediately. This device is designed to assist recovery and should be used exclusively under the instruction of and in consultation with a healthcare professional. DARCO International Inc. does not assume any responsibility for misuse of the product. Dressings or socks can be worn with the DCS™. Walk more slowly than normal. This device is for single-patient use only.

To the patient/user: This device has been designed and manufactured by DARCO International, Inc. to be state of the art for treatment of a specific medical condition. Should any serious incident occur in relation to this device it should be reported to the manufacturer at the contact information listed on this document and the proper authority of the country or state where you are located. When treatment is complete, dispose of this device according to local laws and ordinances.

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