

Instructions For The Single-Use Monofilament

1. Hold the filament by the paper handle.
2. Use a smooth motion to touch the filament to the skin on your foot for 1-2 seconds. Touch alongside of, and NOT directly on, an ulcer, callus, or scar. Push to make the filament bend.
3. Touch the filament in all sites circled on the drawing on back of monofilament.
4. Place a + in the circle if you can feel the filament at the site and a - if you cannot feel the filament.

Monofilaments are a screening device used to assess loss of protective sensation.

Store at room temperature

Manufactured for:

DARCO International
810 Memorial Blvd.
Huntington, WV 25701 | USA
www.darcointernational.com

MFT-PKG-2508.V01

Testing Sites:

